

# Herpes simplex virus type I / type II nucleic acid detection kit (double fluorescence PCR method)

**Product Number: DTK812**

## Shipping and Storage

1. Store at -20°C away from light, with a shelf life of 12 months.
2. Low temperature transportation cannot exceed 4 days; After opening, store in the dark at -20°C without affecting the expiration date. Avoid repeated freeze-thaw cycles, as six freeze-thaw cycles will not affect the detection results.
3. Production date and expiration date: see outer packaging box.

## Component

Component	25T	25T	Main components
qPCR premix (containing enzymes)	400μL	800μL	Tris, KCl, MgCl <sub>2</sub> , dNTPs, Taq enzymes, etc
Primer probe HSV1/HSV2	100μL	200μL	Primer probe
Positive control HSV1/HSV2	50μL	50μL	Plasmids containing target detection gene fragments
Negative control	50μL	50μL	Water treated with diethyl carbonate

## Description

This reagent kit is designed based on the principle of fluorescence PCR technology, with specific primers and Taqman probes designed for herpes simplex virus type I/II. It is detected by a fluorescence PCR detector to achieve the detection of herpes simplex virus type I/II nucleic acid.

## Application

This kit is used for qualitative detection of herpes simplex virus type I/II (HSV1/HSV2) nucleic acid, and is used for auxiliary diagnosis and epidemiological monitoring of herpes simplex virus type I/II infection.

## Applicable instruments

The fully automatic fluorescence PCR detector that has undergone multi-channel calibration needs to include FAM, VIC (HEX), CY5, and ROX detection channels, such as ABI7500, 7500FAST, Bio Rad CFX96, Shanghai Hongshi SLAN-96P and other fully automatic fluorescence PCR detectors.

## Specimen collection

1. Sample types: herpetic fluid, saliva, throat wash, corneal swabs, secretions, cerebrospinal fluid, and other samples.
2. Storage conditions: The collected specimens should be sent for testing in a timely manner. Those tested within 24 hours should be stored at 4°C, and those tested beyond 24 hours should be stored at -70°C, avoiding repeated freezing and thawing.

## Protocol

### 1. Reagent preparation (reagent preparation area)

Melt the components of the reagent kit at 4°C in the dark, mix thoroughly, and centrifuge immediately. Calculate the number of reagents used N (N=number of samples+1 tube of positive control+1 tube of negative control), configure the reaction system mix according to the table below, add it to an appropriate volume of centrifuge tube, mix thoroughly, and centrifuge immediately. Divide it into 20μL PCR reaction tubes/plates and transfer it to the sample processing area.

Component	Volume (μL)
qPCR premix (containing enzymes)	16

**For Research Use Only**

Primer probe HSV1/HSV2	4
Total volume (reaction system mixture)	20

**2. Sample processing (sample processing area)****2.1. Nucleic acid extraction**

Select the appropriate nucleic acid extraction kit to extract nucleic acid, and follow the instructions of the corresponding kit for specific operations.

**2.2. Sampling**

Add 5μL of processed nucleic acid, negative control, and positive control to the PCR reaction tube/plate that has been added to the reaction system mix, resulting in a final volume of 25μL. Cover the tube tightly or seal the membrane, centrifuge at low speed instantly, and amplify with a fluorescence PCR detector.

**3. Amplification testing (nucleic acid amplification area)**

Step	Cycles	Temperature	Time
Pre denaturation	1 cycle	95°C	5min
Denaturation	40 cycles	95°C	10sec
Annealing/extension/fluorescence detection*		55°C	30sec

\*Fluorescence detection at 55°C in step 2, using FAM and HEX/VIC detection channels.

\*The ABI series fluorescence PCR instrument does not select ROX calibration, and the quenching group is selected as None.

**4. Result analysis**

According to the analysis of the image, adjust the start and end values (it is recommended to start from 3-15 and end from 5-20, and adjust the amplification curve of the negative control to be flat or below the threshold line). Click on the analysis button and view the results on the report interface.

**Quality control**

1. Negative control: Ct value>38 or not detected.
2. Positive control: The amplification curve is S-shaped and the Ct value is ≤ 30.
3. The above requirements must be met simultaneously for the same experiment, otherwise this experiment will be considered invalid.
4. Each detection target requires a positive and negative control, and the baseline threshold is adjusted for different targets based on their corresponding negative results.

**Result interpretation**

1. FAM channel detects HSV1, HEX/VIC channel detects HSV2.
2. Negative: Ct value>38 or not detected.
3. Positive: The amplification curve is S-shaped and the Ct value is ≤ 35.
4. Suspicious: The amplification curve shows an S-shaped pattern and 35<Ct value ≤ 38, requiring retesting; If the retest results are consistent, the judgment result is positive.

**Limitations of detection methods**

1. Improper sample collection, transportation, and storage, as well as improper transportation, storage, and configuration of reagents, can all affect experimental results and even lead to false negative results.
2. If there is laboratory contamination, reagent contamination, or sample cross contamination, false positive results may occur.

**Performance indicators of reagent kit**

1. Minimum detection limit:  $1 \times 10^3$  copies/mL.
2. Linear detection range:  $2 \times 10^3 \sim 1 \times 10^8$  copies/mL.
3. Specificity: It can detect all specimens of herpes simplex virus type I/II and does not cross with other types.



## **MEBEP TECH(HK) Co., Limited**

Email: [sales@mebep.com](mailto:sales@mebep.com) Website: [www.mebep.com](http://www.mebep.com)

Tel: +86-755-86134126 WhatsApp/Facebook/Twitter: +86-189-22896756

### **Note**

1. Each stage of PCR operation should be strictly partitioned to avoid cross contamination.
2. The components of the reagent kit should be thoroughly melted and mixed before use, and centrifuged for a few seconds before use.
3. Each component shall not be interchanged with other products or corresponding ingredients of different batch numbers.
4. If the test specimen is not tested in a timely manner, it should be stored at -20°C or -70°C.
5. The processing of samples should strictly follow biosafety regulations.
6. PCR operators should have experience and receive professional training.
7. This kit is only for scientific research use and is not intended for clinical diagnosis.